




# Treatment Times

## In-Office Cryosurgery

**Quick – Effective – Profitable – Reimbursable:**

The Histofreezer® Portable Cryosurgical System is a low cost highly effective cryosurgical agent for the treatment of a variety of benign skin lesions. Ideal for most patients, the Histofreezer® unit is indicated for use on nine type of lesions. The following is a chart summarizing the treatment times for each indications.

Indication	Treatment Time (seconds)	Resolution Rate	Return Visit (maximum treatment - 4 applications)
Verruca Vulgaris	40	94%	2 weeks
Verruca Plana	20	93%	2 weeks
Molluscum Contagiosum	20	100%	2 weeks
Condyloma Acuminata	40	89%	2 weeks
Actinic Keratosis	40 (non-facial)	91%	2 weeks
	15 (facial)	87%	
Acrochordon (Skin Tags)	40	100%	2 weeks
Seborrheic Keratosis	40	95%	2 weeks
Lentigo	40 (non-facial)	99%	2 weeks
	15 (facial)	100%	

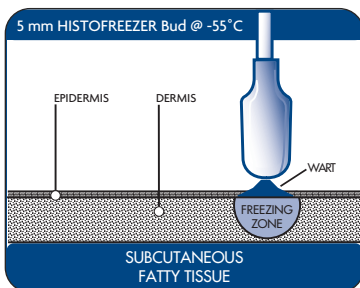

A small ring of healthy tissue around the lesion also be frozen during process. Freezing begins within a few seconds of applying the bud to the lesion and may appear as white discoloration of the skin. The patient experiences a mild tingling/itching sensation and sometimes a slight sting if the region treated is rich in nerve endings.

Skin loses its white color a few minutes after the bud has been removed. Erythema, the same size as the frozen region, then appears. Blistering occurs (possibly blood-filled) within a few days.

Refer to the product insert for a detailed explanation on treatment procedures.

